



Healthy eyes starts with your lids and lashes.

Fresh Eye Lid & Lash spray is the most effective solution for lid & lash health on the market. The spray consists of one of the purest forms of Hypochlorous Acid in the world. It is a natural oxidant produced by the white blood cells within our body and targets a broad spectrum of bacteria that helps to relieve various chronic eye conditions.

How to use your Lid & Lash spray



Wash hands and area around eyes.



Spray 2 sprays directly on the lid and lash margin with eyes closed. Wait for 20 seconds.



Gently wipe the lids and lashes 2-3 times. Apply your daily nutritional eye cream. Repeat daily.



Treatment for:

Dry Eye disease

Blepharitis

Meibomian Gland dysfunction

Styes

Inflammation

Contact lens intolerance

Optometrist instruction

Available from your Optometrist



Your eyes deserve a Spa day every day.

The Eye Spa Eye Therapy Mask contains temperature retaining thermabeads for long-lasting heat or cold therapy. The naturally hydrating beads help to relieve various chronic eye conditions and play a very important part in your at-home therapy to treat these conditions. Non-allergenic and anti-bacterial.

How to use your Eye Therapy Mask



Heat therapy - Heat mask in microwave for approx 20 sec.
Cold therapy - Place mask in freezer for approx 45 min.



Place mask over eyes and fasten strap. Use for approx 10 min or as prescribed.



The mask is reusable and can be hand washed as needed. Slipcover is machine washable.



Treatment for:

Heat therapy

Dry Eye disease

Blepharitis

Meibomian Gland dysfunction

Styes

Cold therapy

Sinus pressure

Tension

Swelling & puffiness

Optometrist instruction

Available from your Optometrist