

# RIPLE-H THERAPY







# FOR OPTIMAL EYE HEALTH









The iMask and Dr Fischer DRY wipes serve as a complimentary aid to medical treatment.

Officially distributed by



Tel: 021 492 9000

Email: info@eyepharma.co.za





EYE-CARE DRY

Hydrating Cleansing Wipes

Optimal Care of Eyelids and Lashes

Complete your treatment regimen with

Professional Ophthalmic-Based Wipes







# Hygienic Cleansing Wipes

Optimal Care of Eyelids and Lashes

#### Exclusive Ophthalmic Technology

Fischer Pharmaceuticals has exclusively developed Dr. Fischer Eve-Care®, ophthalmic based cleansing wipes for hygienic care of the eyelids and lashes. The product undergoes a purification process taking the production standards. to the highest levels. Dr. Fischer Eve-Care® pre-moistened towelettes contain. a soothing ophthalmic solution that is formulated to the natural pH of tears. Dr. Fischer Eve-Care® towelettes employ a special embossed texture technology for enhanced surface cleansing of the eye surrounds. This technology optimized innovation can be applied for both daily and medical usage, ensuring a safer and more effective care.

#### Optimal Ophthalmic Care

Non-ophthalmic based wipes may cause irritation, especially in disordered eyes. Many ocular conditions, such as Blepharitis, may become more severe by irritation; it can be a postponing factor in the healing process, Dr. Fischer Eve-Care® is especially designed to suit a variety of eye conditions. It does not contain preservatives, potential irritants (colorants and fragrances) and has been clinically proven as non-irritating and non-stinging.

#### Safe and Eve-Friendly

Suitable for the most sensitive eyes, Dr. Fischer Eve-Care® Cleansing Wipes effectively dislodge, dissolve and remove mucous deposits and sticky discharge that may form at the base of the eyelids and eyelashes and present a continuing source of infection

Dr. Fischer Eye-Care® Cleansing Wipes have been tested and approved as hypoallergenic, tearless and are enriched with soothing Chamomile extract. Dr. Fischer Eye-Care® is an exceptionally large wipe, soft and embossed for the delicate areas of evelids and lashes.

### Updated to the Latest Ophthalmic Trends and Requirements

Tested and recommended by Ophthalmologists and Dermatologists, Dr. Fischer Eye-Care® is a league of its own in eye health protection.

- · Rinse-Free Tearless



### Complementary Aid to Medical Treatment

#### Common Eve Conditions:

- Blepharitis
- Conjunctivitis
- Hordeolum
- Dacryocystitis
- · Hay Fever Allergy
- Hygienic Care Following Clinical Procedures.

### Advantages of Dr. Fischer Eve-Care®:

- · Soothes the eve surrounds and helps keep the evelids and lashes free of crusts, scales, and residues.
- Non-irritating wipes, helps prevent ocular conditions from worsening. Cares and helps prevent discharge of tear duct purulent secretions.

### Daily Uses of Dr. Fischer Eve-Care®:

- . Contact lens wearers cleans the eyelids and lashes of residues and helps maintain strict lens hygiene.
- Computer Vision Syndrome (CVS) helps relieve irritation and evestrain due to glare and long hours of focusing the eyes.
- Dry eye syndrome helps maintain the function of tear spreading and protects against dryness, Using Dr. Fischer Eve-Care® wipes may assist in reopening the meibomian glands allowing them to secrete oils thereby relieving symptoms.
- . Make-up remover for sensitive eyes gentle and effective cleansing of the evelids and lashes.



# **DIRECTIONS**

## General Cleansing of Eyelids and Lashes



1. Prior to treating the eye area ensure that your hands are clean.



2. Open the sealed pack and remove the wipe.



3. Use a separate wipe for each eye.



- 4.\* Remove all mucous secretions and residues from the eyelids and lashes by wiping in a downwards direction. Make sure to clean in between the lash and corners of the evelids.
- 5. Do not reuse dispose of wipe in a suitable container.

## Warm Compress



1. Prior to treating the eye area ensure that your hands



2. Without removing or opening the pack, soak a Dr. Fischer Eye-Care® Wipe in a container of hot water for a few minutes. Note: Do not heat the wipe if the pack is open.



3. Remove the wipe from the pack and ensure that it is not over hot before use.



4. Place the wipe over the closed eye for a few minutes and then wipe the eyelids and lashes as directed (4\*). Dispose of wipe in a suitable container

To use Dr. Fischer Eye-Care®



## Fischer Pharmaceuticals Over 50 Years of Innovation in Eye Care

Fischer Pharmaceuticals has been involved in the R&D and manufacture of sterile ophthalmic medical preparations since 1965. These products are in routine use by ophthalmologists and medical practitioners in clinics and hospitals.

Dr. Fischer Eve-Care® Wipes are the result of the company's expertise in visual health medication and specialization in sterile ophthalmic preparations.

Fischer Pharmaceuticals innovation in formulation and application of sterile ophthalmic preparations as well as complementary eye-care treatment products has benefited countless numbers of eyes for almost five decades.

### APS - Age Protection System

Fischer Pharmaceuticals manufacture products that incorporate the innovative Dr. Fischer Age Protection System; a state-of-the-art healthcare treatment concept that is reflected in the company's most successful and original product ranges.

